

**select**<sup>TM</sup>  
from NVS

---


Powered by **MPS**<sup>®</sup>  
Medical Pet Shirts



**MANUAL**

# Protective Post op & Recovery Shirt for dogs

Safety and warranty manual for dog owners

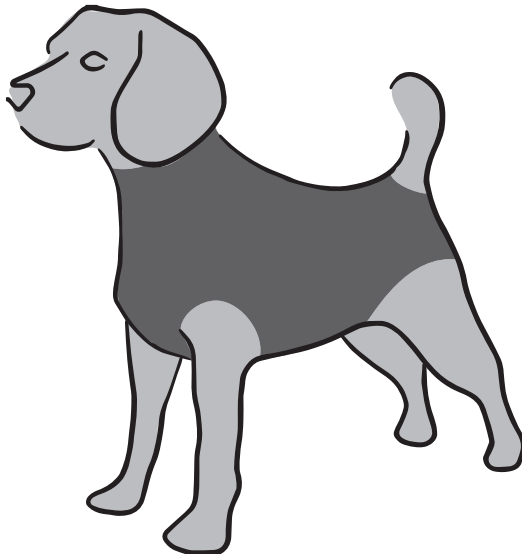

## Product type: Protective Post op & Recovery Shirt for Dogs

Read all safety and product information carefully before using the Protective Post op & Recovery Shirt on your pet. Failure to follow these instructions may cause injury. Protective Post op & Recovery Shirt products are not toys or fashion items. Adult supervision is required during the use of this product. Improper use may lead to hotspots, chafing, skin irritation, inflammation, or infection. In case of emergency or doubt, consult your veterinarian immediately.

### Important safety instructions:

#### 1. Usage.

The Protective Post op & Recovery Shirt should only be used on the advice/consultation of your veterinarian and only after a medical procedure, for medical conditions, and skin problems. In case of emergency or doubt, consult your veterinarian immediately! Do not use the shirt if it is damaged or if it contains loose parts.

#### 2. Supervision.

Your pet must always be under the supervision of a responsible adult while wearing the Protective Post op & Recovery Shirt. Regular checks for cleanliness, friction, constriction, chafing, tripping hazard, or choking hazard are mandatory.

#### 3. Adjustment period.

Allow your pet to get used to the shirt before a surgery and have your veterinarian put the shirt on your pet during anaesthesia. If in doubt, always consult your veterinarian for advice. If your pet tends to chew or gnaw on materials, this product is not recommended. Therefore, always keep your pet under supervision.

#### 4. Unusual behaviour.

In case of unusual behaviour from your pet, which may include but is not limited to walking backward, sneezing, coughing, or incontinence, remove the Protective Post op & Recovery Shirt immediately and consult your veterinarian.

#### 5. Visible issues.

Check for proper fit and any skin irritation daily. Report skin irritation or any other problems to your veterinarian immediately for proper treatment and care.

#### 6. Fit.

Ensure the right fit to avoid safety risks. A shirt that is too large can cause tripping or choking hazards; a shirt that is too small can cause friction or skin irritation. Always measure your pet as described in the manual, which is available through the link at the bottom of the page. The shirt is designed to have a snug fit. It is made of lightweight, breathable, and stretchable material. It may be necessary to stretch the shirt to put it on; this is normal as it is a four-way stretch fabric. Make sure there is always room for 1 to 2 fingers around the legs, chest, neck, and tail. Check at least once a day for any irritation around the legs, chest, neck, and tail, and to ensure that the shirt is in place.

#### 7. Wear time.

Never use the shirt for more than eight (8) consecutive hours unless prescribed by your veterinarian. Check the problem area or surgical zone every day. Remove the shirt every few days, wash it, and let it air dry. For constant protection, we recommend using a second Protective Post op & Recovery Shirt. See the washing instructions in the user manual, available in the link below.

**Disclaimer:** While every effort has been made to ensure that the information in this manual is accurate and complete, National Veterinary Services is not liable for errors in or incorrect use of the manual and the product. We reserve the right to modify the manual.

**Warranty:** Our products are guaranteed against manufacturing defects. In case of a product defect, please contact your veterinarian or the place of purchase.

### Want to know more about this product?

Scan the QR code for detailed information about usage, sizes and more.



[www.nvsweb.co.uk](http://www.nvsweb.co.uk)